

# TICK TWEEZERS

 HASHIR PRODUCTS


Tweeze, Tug, Tick Off!

## SPECIALIZING IN TICK REMOVAL TWEEZERS

Our tweezers are specifically designed and tailored to safely, quickly, and easily remove ticks of all sizes and types




### PRODUCT DETAILS

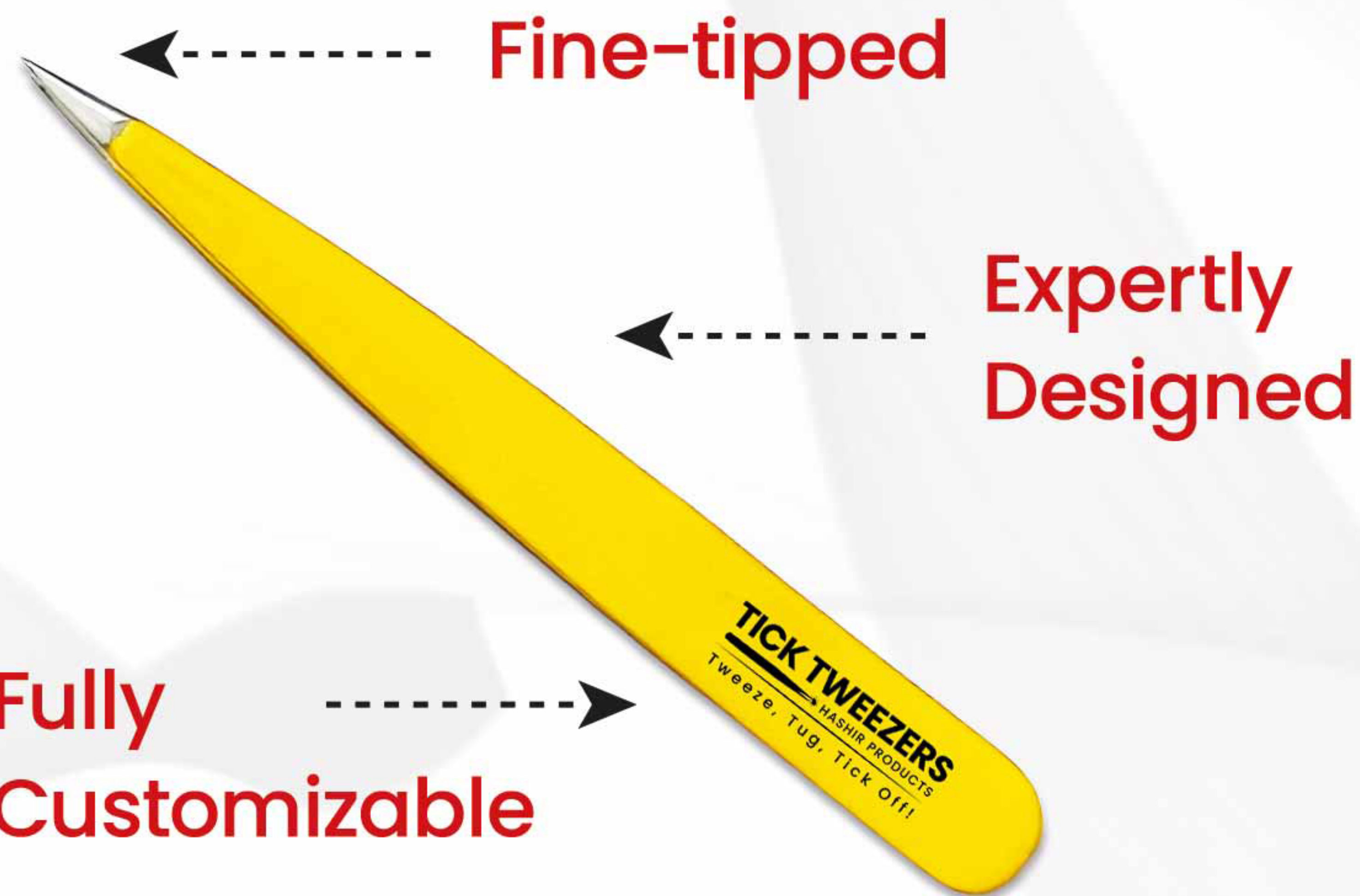

- ✓ Four available sizes: 3", 3.5", 4" and 4.5"
  - ✓ Fully customizable: Add any stamping/text/logo/URL of your choice! (front and back side) - no extra charge!
  - ✓ Color Variation: Available in 7 or more colors
- 
- ✓ Each tweezers is individually packed with a point protector on each of its tip for added safety.
  - ✓ Use fine-tipped tweezers to grasp the tick as close to the skin's surface as possible.
  - ✓ Pull upward with steady, even pressure. Don't twist or jerk the tick; this can cause the mouth-parts to break off and remain in the skin. If this happens, remove the mouth-parts with tweezers.
  - ✓ After removing the tick, clean the bite area, tweezers, and your hands with rubbing alcohol. If you don't have rubbing alcohol, use soap and warm water.

## HOW TO SAFELY REMOVE TICKS

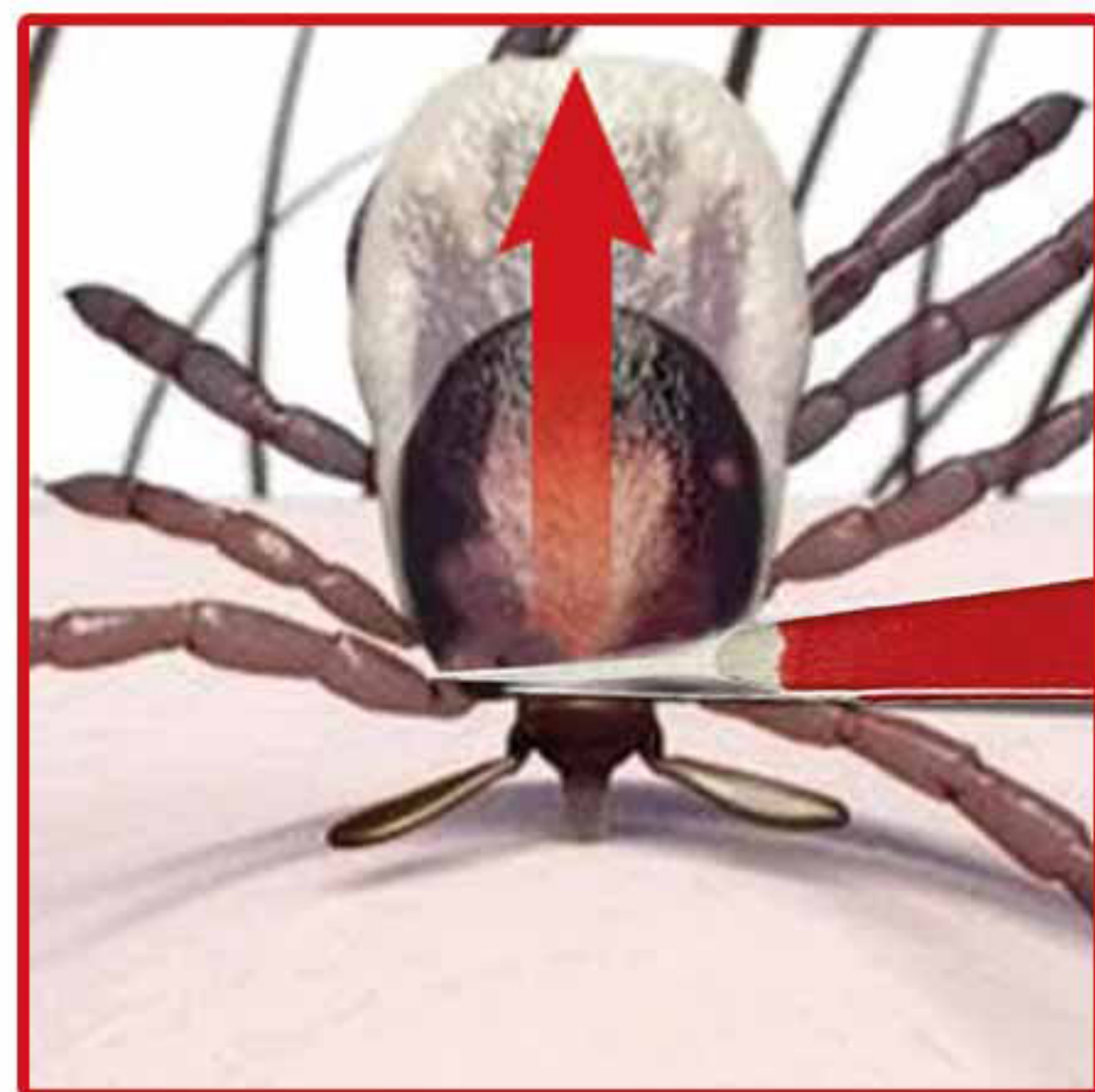

LEARN MORE ABOUT TICKS  
& PREVENTION TIPS ON OUR WEBSITE

Have questions or would like to request more info/samples?  
Contact us today at [sales@ticktweezers.com](mailto:sales@ticktweezers.com)

(914) 434-2690  
[www.ticktweezers.com](http://www.ticktweezers.com)

MAKING TICK REMOVAL EASY!